Jigglers

TALKTOOLS®

www.talktools.com | 888-529-2879 | info@talktools.com
2681 Spruill Avenue, Suite A, Charleston, SC 29405
Designed in Charleston SC. Tips made in the USA. Handle made in China.
Made with USDA approved materials. Patent Pending. Copyright © 2019
ADULT SUPERVISION REQUIRED.

TALKTOOLS® Jigglers

Therapeutic Goals

- Increased Tolerance of Varying Textures/Vibration
- Improved Skills within the Oral Musculature
- Improved Speech Clarity with /oo/ /oh/ /w/ /sh/ /ch/ /m/ /b/ /p/ & more
- Improve motor planning for Oral Placement Therapy Exercises (OPT)
- Improve Feeding

CONTRAINDICATIONS: *Do NOT use this tool with clients who have been diagnosed with seizure disorders that are stimulated by vibration.*

ADDRESSING:

Feeding

Muscle Based
Exercise

Speech

Instructions

Disinfect therapy tools before use

CHOOSING THE MOST APPROPRIATE JIGGLER:

There are three Jigglers to choose from: **Frog, Unicorn, and Sloth**. Each Jiggler addresses specific sensory goals. In some cases you may be using two, or even three, Jigglers during a therapy session, depending upon the needs of the client. Once the sensory goal has been achieved, progress immediately to a muscle-based exercise. The Jigglers are used to prepare the muscle for work; but remember that sensory intervention alone will not improve muscle skills.

All three Jigglers are effective for increasing sensory awareness in the muscles of the jaw and lips, releasing “fixing” or tightening in the muscles of the jaw, and/or satisfying the need for **Temporomandibular Joint (TMJ)** stimulation. *In addition, each individual Jiggler can be used to meet more specific needs:*

FROG

For helping to achieve
Lip Rounding

UNICORN

For helping to achieve
Lip Closure

SLOTH

For helping to achieve
Lip Rounding or a Munch-Chew


NOTE: These can also be used as a reward to help maintain attention, and more.

Frog Jiggler

GOAL #1

To release “fixing” (tightening in the muscles of the jaw) Muscle-Based Exercises, Feeding and Speech. This will satisfy the need for TMJ stimulation.

- A. Place the round part of Frog’s head at the TMJ on the left side of the client’s face and press firmly to release “fixing,” or to give deep pressure to the joint. Hold until the muscles relax, then repeat on the right side of the client’s face. Repeat this process 3 times.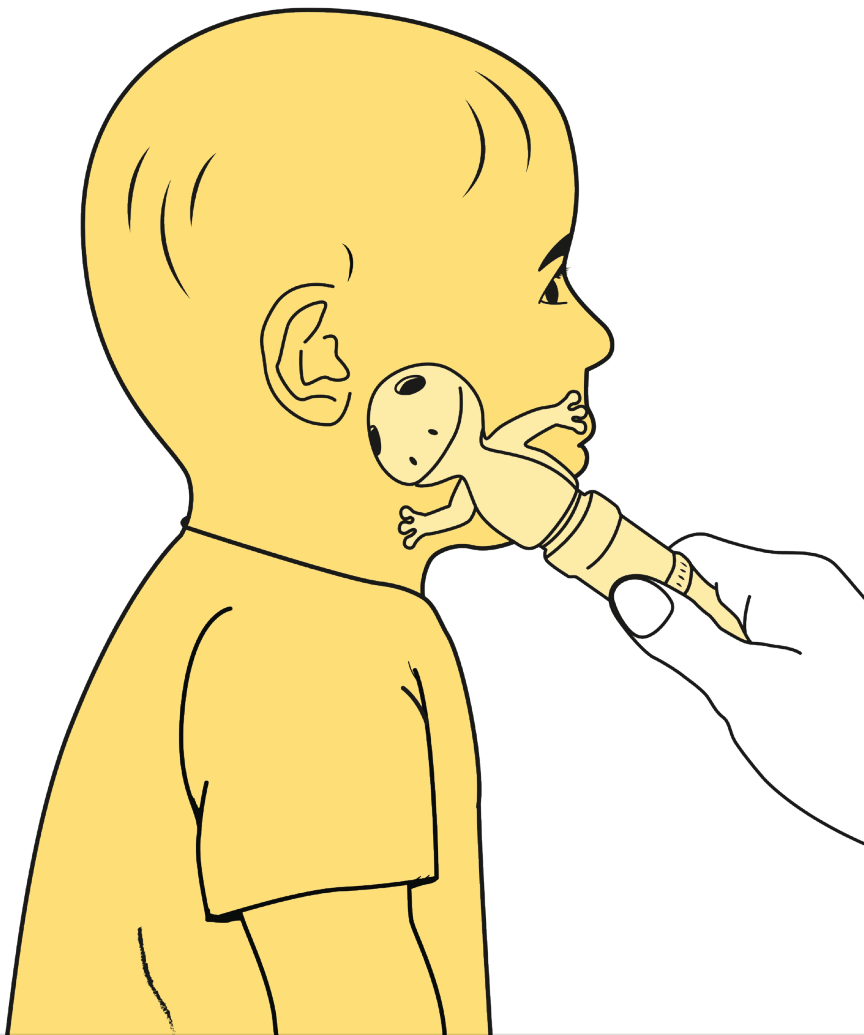

GOAL #2

To increase sensory awareness, using a smooth surface, in the muscles of the jaw and lips prior to the introduction of Muscle-Based Exercises

Speech -/m/ /b/ /p/

Feeding -Spoon Feeding,

OPT -Flat Mouth Horns, Tongue Depressor Exercises

- A. Place the round part of Frog’s head at the TMJ on the left side of the client’s face. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to do so. Stop when you reach the corner of the lips. Repeat on the right side of the client’s jaw.
- B. Repeat this process on the alternating side of the mouth, 3 times. End with firm pressure to the surface of the client’s lips at midline. Remain at this position for 3-5 seconds.

GOAL #3

To stimulate lip rounding

Speech -/oo/ /oh/ /w/ /h/ /sh/ /ch/

OPT -Bubble Blowing and Round Mouth Horns

Feeding -Straw Drinking

- A. Place the round part of Frog’s head at the TMJ on the left side of the client’s face. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to do so. Stop when you reach the corner of the lips. Repeat on the right side of client’s jaw. End by placing the round part of Frog’s head ½ “ from the client’s mouth at the lip midline, and instruct the client to “kiss Frog’s head.” You may squeeze the client’s cheeks to stimulate lip rounding if necessary.
- B. Repeat this technique, stimulating both sides of the mouth, 3 - 10 times.

Unicorn Jiggler

GOAL #1


To increase sensory awareness

- A. Place the round back of Unicorn's head at the TMJ on the left side of the client's face. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to do so. Stop when you reach the corner of the lips. Repeat on the right side of client's jaw
- B. Repeat this process on alternating sides of the mouth, 3 times. End with firm pressure to the surface of the client's lips at midline. Remain at this position for 3-5 seconds.

GOAL #2

To release "fixing" before Muscle-Based Exercises, Feeding and Speech. This will satisfy the need for TMJ stimulation

- A. Place the round back of Unicorn's head at the TMJ on the left side of the client's face and press firmly to release "fixing," or to give deep pressure to the joint. Wait until the muscles relax, then repeat on the right side of the client's face.
- B. Repeat this process 3 times.

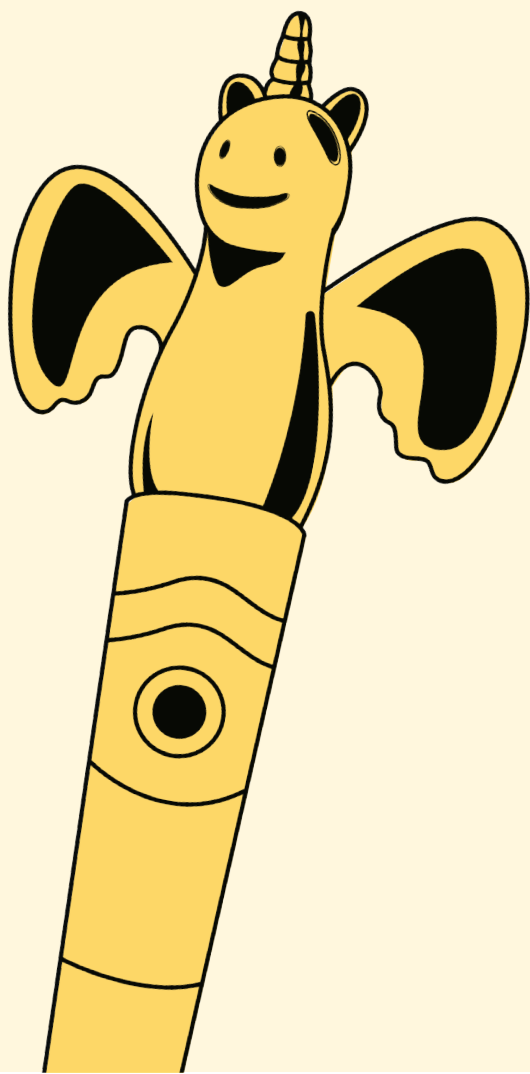GOAL #3

To stimulate lip rounding or as an exercise in itself

Speech -/oo/ /oh/ /w/ /sh/ /ch/

Feeding -Straw Drinking

OPT -Bubble Blowing and Round Mouth Horns

- A. Place the round back of Unicorn's head at the TMJ on the left side of the client's face. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to do so. Stop when you reach the corner of the lips. Repeat on the right side of client's jaw.
- B. Repeat on the right side of the client's jaw. End by placing the top of Unicorn's head ½ " from the client's mouth at lip midline and instruct the client to "kiss Unicorn head." You may squeeze the client's cheeks to stimulate lip rounding if necessary.
- C. Repeat this technique, stimulating both sides of the mouth, 3 - 10 times.

GOAL #4

To increase sensory awareness in the muscles of the jaw and lips to stimulate lip closure prior to the introduction of Muscle-Based Exercises.

Speech -/m/ /b/ /p/

Feeding -Spoon Feeding

OPT -Flat Mouthed Horns and Tongue Depressor

- A. Place the flat part of one of Unicorn's ears between the lips. Instruct the client to press their lips together on the ear, and wait for the lips to close. Hold this position for 3-5 seconds.
- B. Repeat this technique 3-10 times, or until lip closure is consistently achieved.

GOAL #5

To introduce spoon feeding for clients who are fearful

STEP ONE

- A. This exercise can be performed with or without vibration, depending on the client's sensory needs.
- B. Begin by first allowing the client to explore Unicorn, in either the "on" or "off" position, to ensure acceptance of the Jiggler both on and within the mouth.
- C. Once the client accepts the tool, place a small taste of favorite puree on the inside of Unicorn's Wing.
- D. Turn Unicorn sideways, so that the wing is facing the lips in the same position that would be used for spoon feeding. Place the wing on the client's lower lip and wait for the lips to close; you may assist in jaw stability or lip closure if necessary. Once the client achieves lip closure, remove Unicorn's wing from the client's mouth. This should be done with a straight backward motion, not in an upward direction.

- E. Repeat this technique until the client completely accepts the activity.
- F. Place the handle of a spoon on the body of the Jiggler, running lengthwise so that the bowl extends from the base. Put a small amount of a favorite puree in the bowl of spoon, and another on the surface of Unicorn's wing. Feed the child from the Unicorn's wing, then remove the therapy tool and allow the child to swallow. Immediately turn the Unicorn around and feed the child the puree from the surface of the spoon, then remove it and allow the child to swallow. Continue to alternate in this manner, feeding the child with Unicorn's wing followed by the spoon. Gradually eliminate the use of the wing until the child accepts all feedings from the spoon.

STEP TWO

- A. Introduce either lateral spoon placement for lip closure or front spoon placement for lip rounding, depending on the child's needs.

Sloth Jiggler

GOAL #1

To release “fixing” (tightening in the muscles of the jaw) prior to the introduction of Muscle-Based Exercises and/or to satisfy the need for TMJ stimulation

- A. Place the round back of Sloth’s head at the TMJ on the left side of the client’s face and press in firmly to release “fixing,” or to give deep pressure to the joint. Wait until the muscles relax, then repeat on the right side of the client’s face. Repeat this process 3 times.

GOAL #2

To increase sensory awareness in the muscles of the jaw and lips, prior to the introduction of Muscle-Based Exercises

Speech -/m/ /b/ /p/

Feeding -Spoon Feeding

OPT -Flat mouthed horns and Tongue Depressor

- A. Place the round back of Sloth’s head at the TMJ on the left side of the client’s face. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to so. Stop when you reach the corner of the lips.
- B. Repeat on the right side of the client’s jaw. Repeat this process on alternating sides of the mouth, 3 times. End with firm pressure to the surface of the client’s lips at midline. Remain at this position for 3-5 seconds.

GOAL #3


To stimulate lip rounding

Speech -/oo/ /oh/ /w/ /sh/ /ch/

Feeding -Straw Drinking

OPT -Bubble Blowing and Round Mouth Horns

- A. Place the round back of Sloth’s head at the TMJ on the left side of the client’s face.
- B. Use firm, elongated pressure to maintain contact with the surface of the skin while working down the jawbone to the corner of the lips on the left side of the mouth, taking 1-2 seconds to so. Stop when you reach the corner of the lips. Repeat on the right side of the client’s jaw. End by placing the top of Sloth’s head ½ “ from the client’s mouth at lip midline, and instruct the client to “kiss Sloth’s head.” You may squeeze the client’s cheeks to stimulate lip rounding if necessary.
- C. Repeat this technique, stimulating both sides of the mouth, 3-10 times.

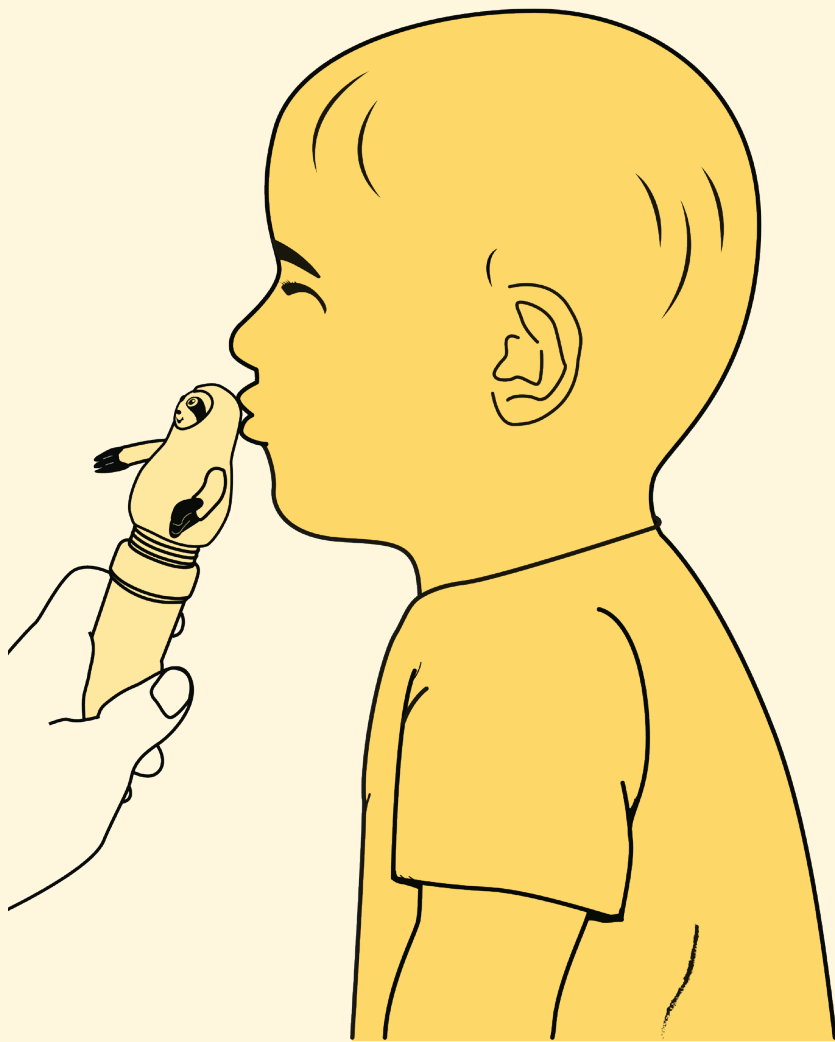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